

# SHARE SATISFACTION CLIO

## USER MANUAL



These weighted kegel balls. Ideal for beginners. Start with the light pink single ball, change the balls as your pelvic muscle strengthen. Swap out the balls from the main units to increase the weight as you progress. The darker the ball the heavier it is, use different combinations to achieve the desired weight.

## SPECIFICATION

Material :	Liquid silicone
Single ball holder :	137mm x 32mm
Double ball holder :	183mm x 34mm
Balls :	18, 32 & 58 grams
Waterproof :	IPX7

## PRODUCT DESCRIPTION AND FEATURES

1. Weighted kegel balls. Ideal for beginners, these silicone toner balls are easy to insert and remove and offer thrilling stimulation and a fantastic pelvic muscle workout for increased control over your orgasm, and his!
2. Made from liquid silicone – soft, safe and reliable.
3. Waterproof - Can be used in the shower, bath or hot tub.

## SAFETY AND DIRECTIONS

1. Remove toy from packaging.
2. Wash with Share Satisfaction toy wash.
3. Apply Share Satisfaction water-based lubricant.
4. Gently insert the ball into the vagina, the long thin tag remains outside for easy removal.
5. This product has a 12month warranty, which covers product replacement for manufacturing defects only. No medical claims are implied or warranted by use of this product.
6. The crossed out wheeled bin on the product indicates that this product should not be treated as household waste.
7. Warning – sold as an adult novelty only, not for medical use. This appliance is not intended for use by children or persons with reduced physical, sensory or mental capabilities. To avoid electric shock. Do not use on swollen or inflamed areas of skin. Discontinue use if discomfort occurs.